

AeroPEP^{Plus}*

Valved Holding Chamber



Two therapies. One device.

The new *AeroPEP Plus** Valved Holding Chamber (VHC) is a combination valved holding chamber and positive expiratory pressure (PEP) device for your COPD and asthma patients. The device can be used as a PEP device alone or in conjunction with the delivery of aerosolized medication from a metered dose inhaler (pMDI) as prescribed by a physician or health care provider.

PEP therapy benefits.

PEP therapy can reduce air trapping in asthma and COPD and can be used as a substitute for pursed-lip breathing. Additionally, PEP therapy can promote bronchial hygiene, improve bronchodilation, and can be used to treat atelectasis.

Advanced valve system.

An adjustable orifice PEP device is built into the chamber exhalation valve. The aerosol delivery and PEP therapy features operate independently and do not interfere with each other.

Six PEP settings.

The new *AeroPEP Plus** VHC features six PEP settings that allow the caregiver to prescribe the appropriate resistance level. A removable adapter to attach a pressure manometer is included to determine an initial setting.

Six PEP Settings



Trudell Medical International

www.trudellmed.com



Combination Valved Holding Chamber and PEP Device




Easy to Use

- Place manometer adapter on to the mouthpiece.
- Connect adapter with tubing to a manometer.
- Adjust the PEP setting to desired PEP level.
- Review instructions with the patient.
- PEP therapy and choosing the PEP setting should be done by a physician or health care provider experienced in the benefits and contraindications associated with PEP therapy.

High Quality Aerosol Delivery

- Aerosol delivery and PEP therapy features operate independently and do not interfere with each other.*

*J Aerosol Med 2002;15(1):104

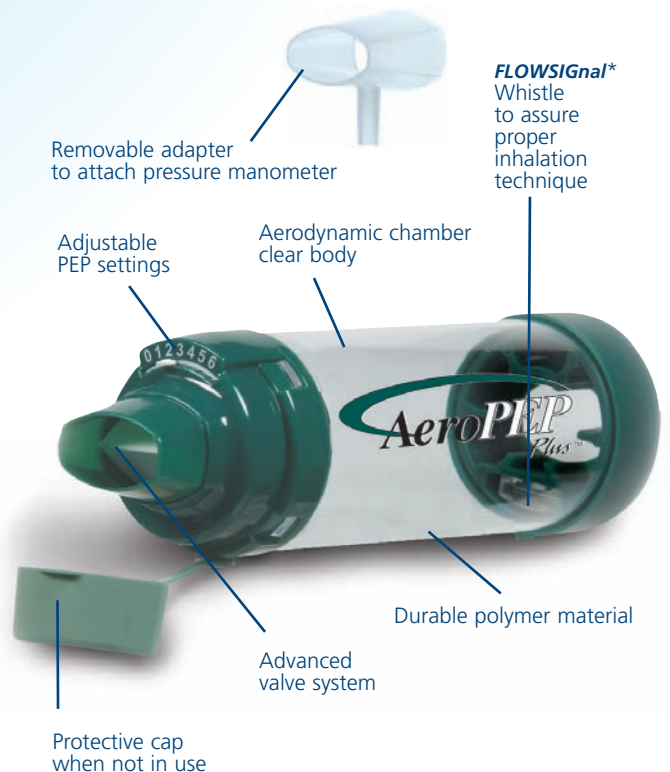

Flow Dynamic Design

Improves respirable dose of medication. Exhalation can be controlled to allow for positive expiratory pressure.



Trudell Medical International*

Trudell Medical International
725 Third Street, London,
Ontario, Canada, N5V 5G4
Phone: +1-519-455-7060
Fax: +1-519-455-7858
www.trudellmed.com

Authorised European Representative:
Trudell Medical International Europe Ltd.
BioCity Nottingham, Pennyfoot Street
Nottingham, NG1 1GF, UK
Tel: +44 115 912 4380
Fax: +44 115 912 4289

