 Directions for Use (Single Patient Use Only) Read instructions prior to use. • Mode D'Emploi (À être utilisé par un seul patient) Lire les instructions avant l'utilisation. • Gebrauchsanleitung (Nur für einen Patienten anwenden) Bitte vor der Anwendung die Gebrauchsanleitung sorgfältig durchlesen • Istruzioni per l'uso (per un singolo paziente soltanto). Leggere attentamente le istruzioni prima dell'uso. • Gebruiksaanwijzingen (uitsluitend voor gebruik bij één patiënt). Lees de instructies vooraleer u het product gebruikt. • Οδηγίες χρήσης (για χρήση σε έναν μόνον ασθενή). Πριν από τη χρήση, διαβάστε τις οδηγίες.

ENGLISH - INTENDED USE

The **AeroPEP Plus** combination Valved Holding Chamber ("VHC") and Positive Expiratory Pressure ("PEP") device can be used as a PEP device alone or in conjunction with the delivery of aerosolized medication from a metered dose inhaler (MDI) as prescribed by a physician or health care provider. **The aerosol delivery and PEP therapy features operate independently and do not interfere with each other.**

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ENGLISH - CONTRAINDICATIONS

The following should be carefully evaluated before a decision is made to initiate PEP therapy:

- Patients unable to tolerate the increased work of breathing (acute asthma, COPD)
- Intracranial pressure (ICP) > 20 mm Hg
- Hemodynamic instability (blood pressure instability)
- Recent facial, oral, or skull surgery or trauma
- Known or suspected tympanic membrane rupture or other middle ear pathology
- Acute sinusitis

- Epistaxis (bleeding nose)
- Esophageal surgery
- Nausea
- Active hemoptysis (bleeding from lungs)
- Untreated pneumothorax (untreated collapsed lung)

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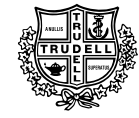
Trudell Medical International



Manufactured by: Trudell Medical International
725 Third St., London, ON, Canada N5V 5G4
E-mail: tmi@trudellmed.com
Website: www.trudellmed.com

Authorized European Representative:
Trudell Medical International Europe Limited
Sutton Place, 49 Stoney Street
Nottingham, United Kingdom NG1 1LX
Phone: +44 (0) 115 959 7971

Copyright © Trudell Medical International 2003. All rights reserved.
* Trade marks of Trudell Medical International.
Covered by U.S. Patent: 5,042,467; 5,848,588; 6,293,279; 6,345,617; 6,435,177. Canadian, U.S. and foreign patents pending.
Printed in Canada.
P/N 101188-001 Rev. A

REFERENCES:
1. "Use of Positive Airway Pressure Adjuncts to Bronchial Hygiene Therapy" AARC Clinical Practice Guideline, Respir Care 1993;38:516-521.
2. "Positive-expiratory-pressure mask therapy: theoretical and practical considerations and a review of the literature." M.J. Mahlmeister, J.B. Fink, G.L. Hoffman, L.F. Fife, Respir Care 1991;36(11):1218-1230.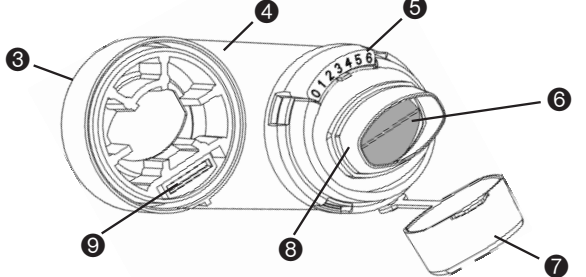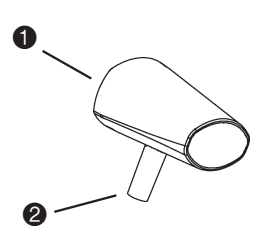

	1	2	3	4	5
ENGLISH	Manometer Adapter	Pressure Port	MDI Adapter	Chamber	PEP Setting Lever
FRANÇAIS	Adaptateur de manomètre	Prise de pression	Adaptateur d'aérosol-doseur	Chambre	Levier de réglage de PEP
DEUTSCH	Druckmesseradapter	Drucköffnung	MDI Adapter	Kammer	PEP Einstellungshebel
ITALIANO	Adattatore del manometro	Portello di pressione	Adattatore MDI	Camera	Levetta di configurazione dei Valori PEP
NEDERLANDS	Manometeradapter	Drukpoort	MDI-adapter	Kamer	Hendel voor instelling van PEP
Ελληνικά	Προσαρμογέας μονομέτρου	Θύρα πίεσης	Προσαρμογέας MDI	Θάλαμος	Μοχλός ρύθμισης PEP

	6	7	8	9
ENGLISH	Inspiratory Valve	Mouthpiece Cap	Mouthpiece	FLOWISGnal® Whistle
FRANÇAIS	Valve inspiratoire	Capuchon de l'embout buccal	Embout buccal	Sifflet FLOWISGnal® Whistle
DEUTSCH	Inhalationsventil	Mundstückkappe	Mundstück	FLOWISGnal® Pfeife
ITALIANO	Valvola d'inspirazione	Cappuccio del boccaglio	Boccaglio	Dispositivo di fschio FLOWISGnal®
NEDERLANDS	Ademhalingsklep	Dop op mondstuk	Mondstuk	FLOWISGnal®-fluitje
Ελληνικά	Βαλβίδα εισπνοών	Πώμα εξαρτήματος στόματος	Εξάρτημα στόματος	Σφυρίχτρα FLOWISGnal®

Valved Holding Chamber (VHC) and Positive Expiratory Pressure (PEP) combination device

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