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**REF** 01-400202 : Light  
01-400203 : Medium  
01-400204 : Heavy




**WWW.MOVES-YOU.COM**

MVS In Motion  
Westdijk 150  
2830 Tissett (Willebroek)  
Belgium - Europe  
www.mvs-in-motion.com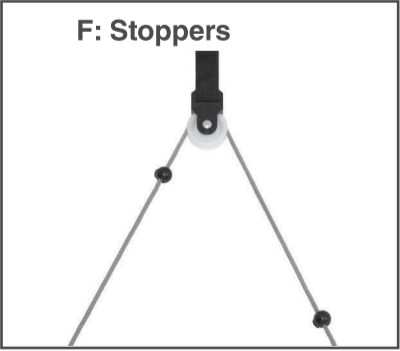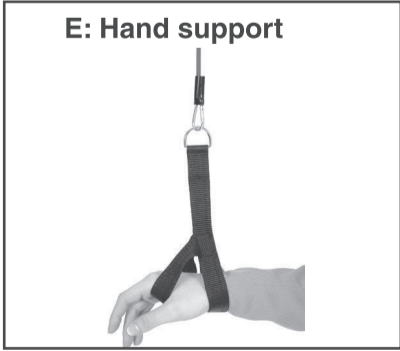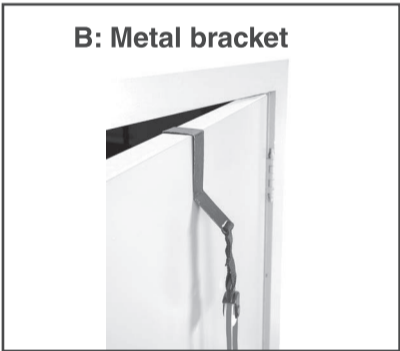

V202006

## EN | SAFETY PRECAUTIONS & INSTRUCTIONS FOR USE Medical Device

### Indications

- To increase upper body range of motion and improve strength;
- For rehabilitation after shoulder injury or cerebrovascular accident (stroke).

### Contraindications

- Do not use in case of fractures;
- Do not use in case of severe oedema.

**Intended user:** lay person and healthcare professional.

### Warnings

- Maximum load: 50kg (110 lbs) for ROPE Pulleys, 20kg (44 lbs) for TUBE Pulleys;
- Shoulder TUBE Pulley contains latex which may cause allergic reactions.

### Precautions

- In case of unusual swelling, skin discoloration or any other discomfort, stop using the device and consult a healthcare professional;
- The door to which the device is attached should be of standard thickness, closed and locked before using the device;
- Device should not be attached to a glass door;
- Persons with heart disease should consult a doctor before using this de-vice;
- Devices with detachable hand support [01-400103]: do not tie the hand support around hand or wrist in a way that might impair blood circulation.

### Care and cleaning

- Shoulder ROPE Pulley: Clean with lukewarm water and mild soap and let dry;

--	--	--	--	--	--

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- Shoulder TUBE Pulley: keep dry and away from sunlight. Do not clean tube.

### Components

The pulley systems include the following components:

- Shoulder ROPE Pulley – Door Anchor [01-400101/3/5]
- 1 x Door anchor strap with pulley system
- 2 x Stoppers on rope
- 1 pair of Notch handles attached to rope [01-400101]
- 1 pair of Detachable Hand Support attached to rope [01-400103]
- 1 pair of Wooden handles attached to rope [01-400105]
- Shoulder ROPE Pulley – Metal Bracket [01-400102]
- 1 x Metal bracket for attachment to door
- 2 x Stoppers on rope
- 1 pair of Wooden handles attached to rope
- Shoulder TUBE Pulley – Door Anchor [01-400202/3/4]
- 1 x Door anchor strap
- 1 x Latex tube with light (yellow), medium (red) or heavy (green) re-sistance

- 1 pair Padded soft handles attached to Tube

### General Instructions

- Use standard-width non-glass door
- Attach strap or bracket as follows:
  - Door anchor strap* (A): With door open, place strap over door. Close door firmly and lock it. Unfold rope. Pull on rope to ensure door strap is properly in place and door is firmly locked.
  - Metal bracket* (B): With door open, place bracket onto top of door and push down firmly. Close door and lock it. Unfold rope. Pull on rope to

--	--	--	--	--	--

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ensure bracket is properly in place and door is firmly locked.

- Adjust length of ROPE pulley line as follows:
  - Notch handle* (C): To shorten pull unwanted length through holes on plastic adjustment piece and handle. Tie knot. Test cord to be sure knot is tight before each use of pulley.
  - Wooden handle* (D): To shorten pull unwanted length through grommet hole in handle and tie knot. Test cord to be sure knot is tight before each use of pulley.
  - Hand support* (E): Attachable and detachable by including carabiners.
- For Shoulder ROPE Pulleys: Use stoppers on rope to adjust for allowed range of motion. Press button and slide rope through hole in bead to raise or lower bead on rope (F).
- TUBE pulley comes with padded soft handles (G).
- Report any serious incident (death, serious deterioration of health, seri-ous public health threat) to the Manufacturer and your Competent Authority.

--	--	--	--	--	--

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