

TipiCallos

Contains Salicylic Acid

Cure-corn plasters

Tipicallos Plasters, with a central disc in soft polyethylene foam, perform a smoothing and protective action to relieve pain in corns and calluses caused by rubbing and pressure in these areas. Additionally, Tipicallos dressings include in their composition Salicylic Acid, helping to the softening, exfoliation and removal of annoying corns and calluses.

DIRECTIONS FOR USE: Before applying the plaster, carefully clean and dry the area to be protected to ensure perfect adhesion and effectiveness. Remove the corn plaster from the paper, and position it centrally over the area to be protected. Apply a new corn plaster daily. Avoid using it if there are rednesses, open wounds, cuts or cracks.



Ref. 6102.2 · Tipicallos 10 units
Ref. 6102.3 · Tipicallos display with 14 boxes

Distributed by: