


Bischoff & Bischoff


Instruction manual

TS-1



Enjoy mobility.

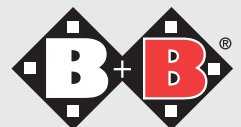

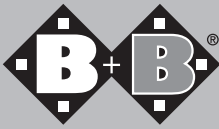

TS-1

(commode wheelchair)

Dear customer,

with the purchase of this B+B commode wheelchair you are now the owner of a high-quality German product, thousands of which are already in professional use.


This instruction manual will familiarise you with the use and maintenance of your B + B commode wheelchair and make your life safer.

We hope you enjoy our product and are more mobile with it.

1. Attention - safety instructions! **(Risk analysis as per DIN EN 13485)**

- Max. load capacity 120 kg
- Never use the commode wheelchair without an assistant.
- Use your walking aid on even and horizontal surfaces.
- Always lock both brakes when using your wheelchair.
- Do not lean out to the side. Danger of tipping!
- Make sure that the seat-back screws are tight.
- The commode wheelchair must not be used by unauthorised persons (particularly children).

Commode wheelchairs are recommended if the patient cannot use or access toilette because of the disability.

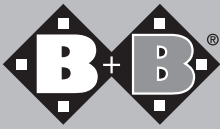

- Removable seat back
- Back cover
- Adjustable arm rest with padding
- Locking pin for arm rest
- Seat cushion
- Fixation screw for back rest
- Locking lever for leg supports
- Commode with cover
- Leg supports
- Back steering wheel, braked
- Front steering wheel, unbraked



Fig. 1

2. Parking brake

The two rear steering wheels with brakes prevent the wheelchair from moving accidentally.

- Setting brakes: press brake lever down to the limit stop with your foot (Fig. 2)
- Releasing brakes: pull the seat upwards with your toe (Fig. 2).

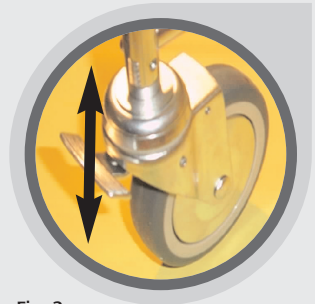


Fig. 2

3. Leg supports

To make it as easy as possible to enter and leave your B + B commode wheelchair, the leg supports can simply be swung to the side out of the way. It is not necessary to remove them.

- Press the locking lever slightly inwards (Fig. 3) to release the lock.
- Swing the leg supports outwards.
- If necessary, fold the foot plates up first.
- To restore the original status, just swing the leg supports forward and lock them in place.
- To remove the leg supports proceed as follows:
 - Press the locking lever slightly inwards (Fig. 3) to release the lock.
 - Swing the leg support outwards.
 - Pull the leg support out upwards.



Fig. 3



Danger of accident!

Be careful not to stand on the foot plates and/or the foot supports when standing.



Fig. 4

4. Commode seat

Removing commode seat:

- Pull front edge of seat up from the base.
- Lift commode seat upwards.

Positioning commode seat:

- Slide back edge of seat with the open corners to the back.
- Press front rounded edge of seat down to the stop. Seat must lock into position.
- Make sure that it is locked.

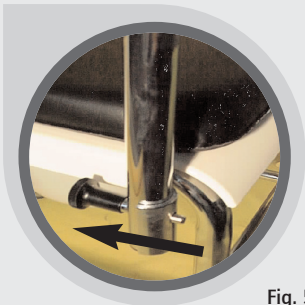


Fig. 5

5. Arm rests

The arm rests can be folded down.

- To fold them down, pull out the locking button and fold the arm rest down (Fig. 4).
- To return it to its original position fold the arm rest up to its original position until you hear the locking pins click into place.




Attention!

Make sure that there are no persons behind your commode wheelchair when adjusting the arm rests.

6. Technical data

Seat width:	45 cm
Seat depth:	44 cm
Length:	96 cm
Height:	97 cm
Width:	56 cm
Seat to arm rests:	22 cm
Seat back rest:	34 cm
Max. user weight:	120 kg
Weight:	15,3 kg

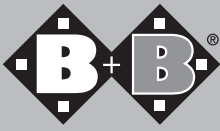

Warranty certificate

(Please note our general terms and conditions)

Model: TS-1 commode wheelchair

Serial number:*

(to be entered by dealer)

Dealer:

Date and stamp

* See the side frame for information stickers

Bischoff & Bischoff GmbH

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 (0)700/6000 7070 • Fax: +49 (0)700/6000 8080

www.bischoff-bischoff.com

info@bischoff-bischoff.de

